

# NEOMED\* TAMPER EVIDENT CAP FOR ENFit® SYRINGES



1

Insert ENFit® Syringe into Tamper Evident Cap and rotate clockwise while pushing downwards (into cap).



2


Verify installation by rotating the cap until snug. This typically requires less than 1/4 turn.



3


Visually inspect Tamper Evident Cap for evidence that the syringe has been previously used or tampered with, such as:

- Missing components (outer cover A)
- Visible damage

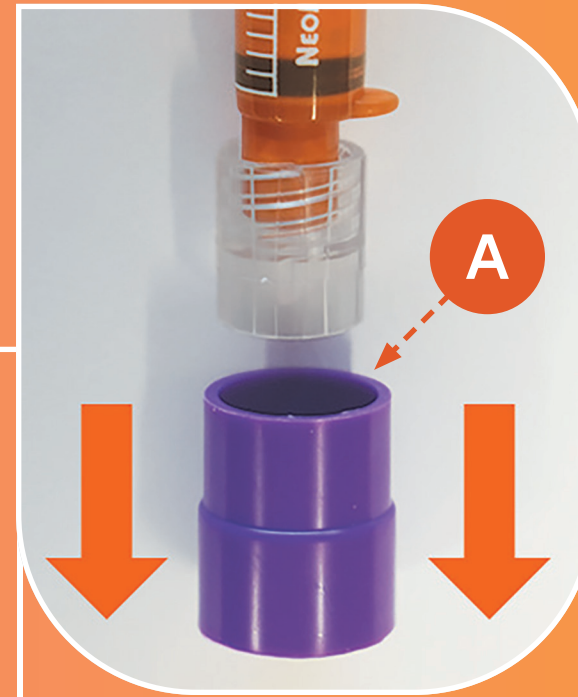

4

Remove the Tamper Evident Cap's outer cover (A) by pulling it directly away from the attached syringe.

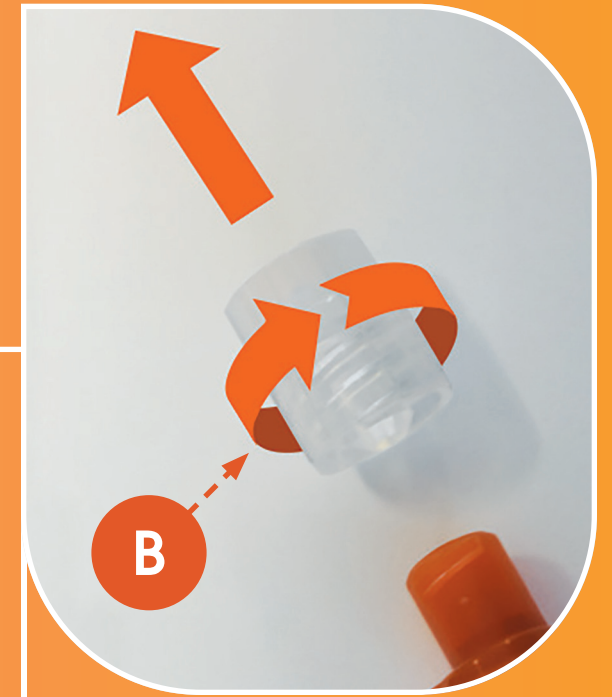

5

Remove tip cap (B) from syringe by twisting it counter-clockwise.



- Damaged or missing Tamper Evident outer cover (A) indicated potential tampering and syringe should be discarded.
- This is a single use device. Do not re-use or re-attach the tip cap (B) after it has been removed from the syringe.
- Properly installed cap remains sealed on syringe until removed.
- Do not use if package is opened or damaged.

There are inherent risks in all medical devices. Please refer to the product labeling for indications, cautions, warnings and contraindications by visiting <https://eifu.avanos.com/AVA/all>. Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.

AVANOS\*

[avanosmedicaldevices.com](https://www.avanosmedicaldevices.com)