

NEOMED* TRANSFER LID USED WITH ENFit® SYRINGE



1

Always use aseptic technique.

Screw the Transfer Lid onto a storage container.



2

Connect an ENFit® syringe to the Transfer Lid port.



3

Add content with the container upright

OR

Remove content with the container inverted.



4

Return the container to an upright position, remove the syringe, then close the Transfer Lid.



Cautions: Do not autoclave or re-use Transfer Lid as this may increase the risk of leakage or biological infection. To avoid spillage, keep container in upright position when adding and storing contents. Keep container inverted when removing contents.

There are inherent risks in all medical devices. Please refer to the product labeling for indications, cautions, warnings and contraindications by visiting <https://eifu.avanos.com/AVA/all>. Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.

AVANOS*

[avanosmedicaldevices.com](https://www.avanosmedicaldevices.com)