

ENTERAL FEEDING WITH ENFit®



NEOMED* SAMPLING STRAW WITH ENFit® CONNECTOR

1. Wash hands per facility protocol
2. Attach straw to syringe
3. Place end of straw into container or cup
4. Draw up desired volume
5. Remove any air bubbles
6. Remove straw and cap syringe until ready for use




NEOMED TRANSFER LID ENFit® SYRINGE COMPATIBLE

1. Always use aseptic technique
2. Screw the Transfer Lid onto a storage container
3. Connect an ENFit® syringe to the Transfer Lid port
4. Add content (container upright) or remove content (container inverted)



NEOMED COLOSTRUM AND BREASTMILK CONTAINER WITH TRANSFER LID ENFit® SYRINGE COMPATIBLE

1. Wash hands and remove container from package. Avoid touching inside the bottle and lid
2. Screw container to breast shield and express milk
3. When finished, remove from breast shield and securely attach Transfer Lid onto the storage container
4. Label the container immediately
5. Connect a NEOMED ENFit® Syringe to the Transfer Lid port
6. Add content (container upright) or remove content (container inverted)
7. Secure tethered plug to syringe port

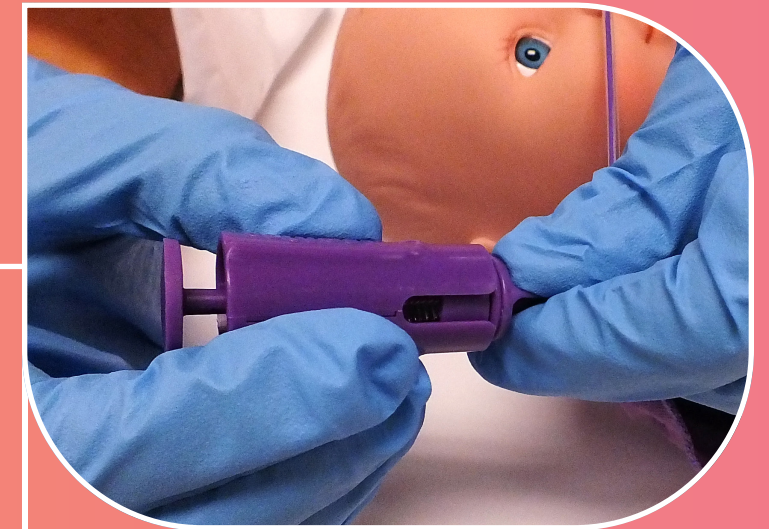

CLEANING THE ENFit® CONNECTOR

1. Remove closure cap from the hub
2. Insert Cleaning Tool plug into the hub. Do not over insert the plug as this may lead to difficulty removing the Cleaning Tool from the hub
3. Slide the Cleaning Tool over the hub to engage internal bristles
4. Rotate the Cleaning Tool to dislodge debris
5. Rotate the Cleaning Tool clockwise at least 360° to sweep debris to the bottom of the hub
6. Align the Cleaning Tool and gently push bristles through bottom of hub to eject debris
7. Remove expelled debris per hospital protocol
8. Carefully remove the Cleaning Tool



- To avoid spillage, keep container in upright position when adding and storing contents. Keep container inverted when removing contents.
- For syringe sizes 6 ml and below, ensure the moat is free of fluids.

There are inherent risks in all medical devices. Please refer to the product labeling for indications, cautions, warnings and contraindications by visiting <https://eifu.avanos.com/AVA/all>. Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.