

NEOMED* ENFit® CLEANING TOOL

NEOMED* Cleaning Tool is used to remove debris from a NEOMED Feeding Tube or Extension Set with an ENFit® male connector for Neonates and Pediatrics.



SCAN TO WATCH ENFIT® CLEANING TOOL VIDEO

SCAN ME



1 Always use aseptic technique. Remove closure cap from the hub.




2 Insert Cleaning Tool plug into the hub. Do not over insert the plug as this may lead to difficulty removing the Cleaning Tool from the hub.



3 Slide the Cleaning Tool over the hub to engage internal bristles.



4 Rotate the Cleaning Tool to dislodge debris. Rotate the Cleaning Tool clockwise at least 360° to sweep debris to the bottom of the hub.

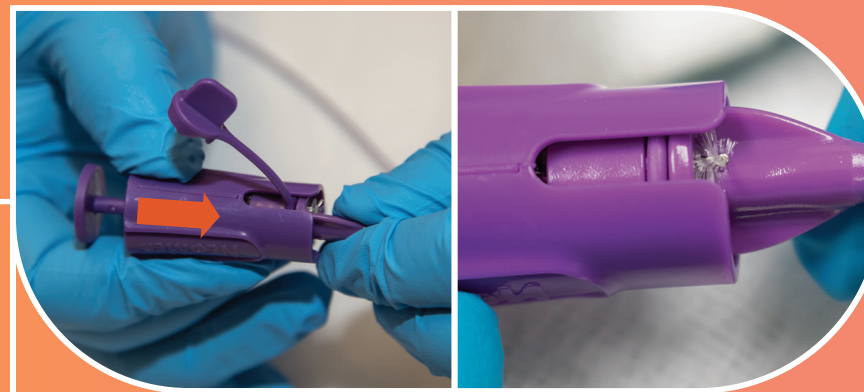

5 Align the Cleaning Tool and gently push bristles through bottom of hub to eject debris. Remove expelled debris per hospital protocol.



6 Carefully remove the Cleaning Tool and dispose as required by hospital protocol and local or national regulations.

⚠ Cautions: Do not autoclave or re-use the NEOMED Cleaning Tool as this may increase the risk of biological infection. Ensure the plug stays engaged during use.

There are inherent risks in all medical devices. Please refer to the product labeling for indications, cautions, warnings and contraindications by visiting <https://eifu.avanos.com/AVA/all>. Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.

Rx only

AVANOS*

[avanosmedicaldevices.com](https://www.avanosmedicaldevices.com)

*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates. © 2018-2024 AVNS. All rights reserved. COPY-07668